rodger

Rodger BAS



- 🇬🇧 What's in the box?
- 🇧🇪 Wat zit er in de doos?
- 🇩🇪 Was ist in der Verpackung enthalten?
- 🇫🇷 Que contient la boîte ?
- 🇪🇸 ¿Qué hay en la caja?
- 🇮🇹 Cosa contiene la confezione?
- 🇨🇷 Co je v krabici?
- 🇵🇱 Co znajduje się w pudełku?
- 🇩🇰 Hvad er der i æsken?
- 🇸🇪 Vad finns i lådan?
- 🇳🇴 Hva er i esken?
- 🇫🇮 Mitä laatikko sisältää?
- 🇬🇷 Τι περιέχει το κουτί;
- 🇵🇹 O que está na caixa?
- 🇷🇺 Ce se află în cutie?
- 🇧🇲 Каково има в кутията?
- 🇭🇺 Mi van a dobozban?
- 🇱🇹 Kas yra dėžutėje?
- 🇮🇹 Što se nalazi u kutiji?
- 🇧🇸 Mis on karbis?
- 🇮🇸 Kas ir kasté?
- 🇲🇩 X'hemm fil-kaxxa?
- 🇮🇸 Kaj je v škati?
- 🇹🇷 Kutunun içinde ne var?

- 🇬🇧 Installation with the plug.
- 🇧🇪 Installatie met de stekker.
- 🇩🇪 Installation mit dem Stecker.
- 🇫🇷 Installation avec la prise.
- 🇪🇸 Instalación con el enchufe.
- 🇮🇹 Installazione con la spina.
- 🇨🇷 Instalace se zástrčkou.
- 🇵🇱 Instalacja z wtyczką.
- 🇩🇰 Installation med stikket.
- 🇸🇪 Installation med kontakten.
- 🇳🇴 Installasjon med støpselet.
- 🇫🇮 Asennus pistokkeella.
- 🇬🇷 Εγκατάσταση με το φις.
- 🇵🇹 Instalação com a ficha.
- 🇷🇺 Instalare cu ștecherul.
- 🇧🇲 Инсталация с щепсела.
- 🇭🇺 Telepítés csatlakozódugóval.
- 🇱🇹 Montavimas su kištuku.
- 🇮🇹 Instalacija s utikačem.
- 🇧🇸 Paigaldamine pistikuga.
- 🇮🇸 Ustáðisana ar kontaktdakðu.
- 🇲🇩 Installazzjoni bil-plagg.
- 🇮🇸 Namestitev z vtičem.
- 🇹🇷 Fiş ile kurulum.

- 🇬🇧 Plug the alarm into the power outlet.
- 🇧🇪 Stop de wekker in het stopcontact.
- 🇩🇪 Stecken Sie den Wecker in die Steckdose.
- 🇫🇷 Branchez le réveil sur une prise électrique.
- 🇪🇸 Conecte el despertador a la toma de corriente.
- 🇮🇹 Collegare la sveglia alla presa di corrente.
- 🇨🇷 Zapojte budík do elektrické zásuvky.
- 🇵🇱 Podłącz budzik do gniazdka elektrycznego.
- 🇩🇰 Sæt vækkeuret i stikkontakten.
- 🇸🇪 Anslut väckarklockan till eluttaget.
- 🇳🇴 Koble vekkerklokken til stikkontakten.
- 🇫🇮 Kytke herätyskello pistorasiaan.
- 🇬🇷 Συνδέστε το ξυπνητήρι στην πρίζα.
- 🇵🇹 Ligue o despertador à tomada elétrica.
- 🇷🇺 Introduceți ceasul deșteptător în priză.
- 🇧🇲 Включете будилника в контакта.
- 🇭🇺 Csatlakoztassa az ébresztőórát a konnektorba.
- 🇱🇹 Įjunkite žadintuvą į elektros lizdą.
- 🇮🇹 Uključite budilicu u utičnicu.
- 🇧🇸 Ûhendage áratuskell pistikupeassa.
- 🇮🇸 Pievienojiet modinātāju elektrības kontaktligzdai.
- 🇲🇩 Qabbad l-allarm mas-sokit tad-dawl.
- 🇮🇸 Priklučite budilko v električno vtičnico.
- 🇹🇷 Alarm saatini prize takın.

- 🇬🇧 Turn on the alarm.
- 🇧🇪 Zet de wekker aan.
- 🇩🇪 Schalten Sie den Wecker ein.
- 🇫🇷 Allumez le réveil.
- 🇪🇸 Encienda el despertador.
- 🇮🇹 Accendere la sveglia.
- 🇨🇷 Zapněte budík.
- 🇵🇱 Włącz budzik.
- 🇩🇰 Tænd vækkeuret.
- 🇸🇪 Sätt på väckarklockan.
- 🇳🇴 Slå på vekkerklokken.
- 🇫🇮 Kytke herätyskello päälle.
- 🇬🇷 Ενεργοποιήστε το ξυπνητήρι.
- 🇵🇹 Ligue o despertador.
- 🇷🇺 Porniți ceasul deșteptător.
- 🇧🇲 Включете будилника.
- 🇭🇺 Kapcsolja be az ébresztőórát.
- 🇱🇹 Įjunkite žadintuvą.
- 🇮🇹 Uključite budilicu.
- 🇧🇸 Lülitage áratuskell sisse.
- 🇮🇸 Ieslēdziet modinātāju.
- 🇲🇩 Ixghel l-allarm.
- 🇮🇸 Vklópíte budilko.
- 🇹🇷 Çalar saati açın.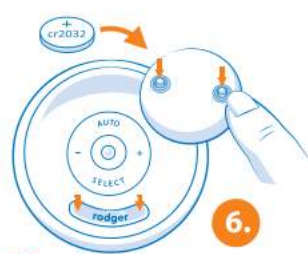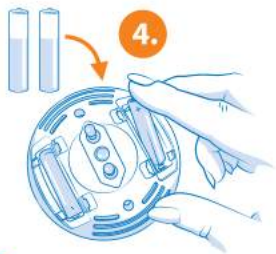

- Installation with batteries.
- Installatie met batterijen.
- Installation mit Batterien.
- Installation avec des piles.
- Instalación con baterías.
- Installazione con batterie.
- Instalace s bateriemi.
- Instalacja z bateriami.
- Installation med batterier.
- Installation med batterier.
- Installasjon med batterier.
- Asennus paristoilla.
- Εγκατάσταση με μπαταρίες.
- Instalação com baterias.
- Instalare cu baterii.
- Инсталация с батерии.
- Telepítés elemekkel.
- Montavimas su baterijomis.
- Instalacija s baterijama.
- Paigaldamine patareidega.
- Uzstādīšana ar baterijām.
- Installazzjoni bil-batteriji.
- Namestitev z baterijami.
- Piller ile kurulum.

- Turn on the alarm.
- Zet de wekker aan.
- Schalten Sie den Wecker ein.
- Allumez le réveil.
- Encienda el despertador.
- Accendere la sveglia.
- Zapněte budík.
- Włącz budzik.
- Tænd vækkeuret.
- Sätt på väckarklockan.
- Slå på vekkerklokken.
- Kytke herätyskello päälle.
- Ενεργοποιήστε το ξυπνητήρι.
- Ligue o despertador.
- Porniți ceasul deșteptător.
- Включете будилника.
- Kapcsolja be az ébresztőórát.
- Įjunkite žadintuvą.
- Uključite budilicu.
- Lülitage äratuskell sisse.
- Ieslēdziet modinātāju.
- Ixgħel l-allarm.
- Vklópote budíliko.
- Çalar saati açın.

- Insert the battery into the transmitter.
- Plaats de batterij in de zender.
- Legen Sie die Batterie in den Sender ein.
- Insérez la pile dans l'émetteur.
- Coloque la batería en el transmisor.
- Inserire la batteria nel trasmettitore.
- Vložte baterii do vysílače.
- Włóż baterię do nadajnika.
- Sæt batteriet i senderen.
- Sätt batteriet i sändaren.
- Sett batteriet i senderen.
- Aseta paristo lähettimeen.
- Τοποθετήστε τη μπαταρία στον πομπό.
- Coloque a bateria no transmissor.
- Introduceți bateria în emițător.
- Поставете батерията в предавателя.
- Helyezze be az elemet az adóba.
- Įdėkite bateriją į siųstuvą.
- Umetnite bateriju u odašiljač.
- Sisestage patarei saatjasse.
- Ievietojiet bateriju raidītājā.
- Daħhal il-batterija fit-trasmettitur.
- Vstavite bateriu v oddajnik.
- Pili vericiye yerleştirin.

- Pair the transmitter with the alarm clock.
- Koppel de zender met de wekker.
- Koppeln Sie den Sender mit dem Wecker.
- Appairez l'émetteur avec le réveil.
- Empareje el transmisor con el despertador.
- Associare il trasmettitore alla sveglia.
- Spárujte vysílač s budíkem.
- Sparuj nadajnik z budzikiem.
- Par senderen med vækkeuret.
- Parkoppla sändaren med väckarklockan.
- Par senderen med vekkerklokken.
- Yhdistä lähetin herätyskelloon.
- Συνδέστε τον πομπό με το ξυπνητήρι.
- Emparelhe o transmissor com o despertador.
- Asociați emițătorul cu ceasul deșteptător.
- Сдвоете предавателя с будилника.
- Párosítsa az adót az ébresztőórával.
- Susiekite siųstuvą su žadintuvu.
- Uparite odašiljač s budilicom.
- Siduge saatja äratuskellaga.
- Savienojiet raidītāju pāri ar modinātāju.
- Qabbad it-trasmettitur mal-arlogg tal-allarm.
- Seznajte oddajnik z budíliko.
- Vericiyi çalar saat ile eşleştirin.

8. Connect to the sensor pants.
- Aansluiten op het sensorbroekje.
- An die Sensorhose anschließen.
- Connecter au pantalon capteur.
- Conectar al pantalón con sensor.
- Collegare ai pantaloni con sensore.
- Pripojte k senzorovým kalhotkám.
- Podłączyć do spodenek z czujnikiem.
- Tilslut til sensorbukserne.
- Anslut till sensorbyxorna.
- Koble til sensorbuksene.
- Liitä sensorihousuihin.
- Συνδέστε με το παντελόνι αισθητήρα.
- Ligar às calças com sensor.
- Conectati la pantalonii cu senzor.
- Свържете към сензорните гащички.
- Csatlakoztassa az érzékelős nadrághoz.
- Prijunkite prie sensorinių kelnaičių.
- Spojite na senzorske hlačiće.
- Ühendage sensorpükstega.
- Pievienojiet sensora biksītēm.
- Qabbad mal-qalziet bis-sensor.
- Priključite na senzorske hlačke.
- Sensörlü eğitim küloduna bağlayın.

9. Turn on the alarm clock.
- Zet de wekker aan.
- Schalten Sie den Wecker ein.
- Allumez le réveil.
- Encienda el despertador.
- Accendere la sveglia.
- Zapněte budík.
- Włącz budzik.
- Tænd vækkeuret.
- Slå på väckarklockan.
- Slå på vekkerklokken.
- Kytke herätyskello päälle.
- Ενεργοποιήστε το ξυπνητήρι.
- Ligue o despertador.
- Porniți ceasul deșteptător.
- Включете будилника.
- Kapcsolja be az ébresztőórát.
- Ijunkite žadintuvą.
- Uključite budilicu.
- Lülitage äratuskell sisse.
- Ieslēdziet modinātāju.
- Ixgħel l-arlogg tal-allarm.
- Vklopite budilko.
- Çalar saati açın.

10. Volume control
- Volumeregeling
- Lautstärkeregelung
- Réglage du volume
- Control de volumen
- Controllo del volume
- Ovládání hlasitosti
- Regulacja głośności
- Lydstyrkekontrol
- Volymkontroll
- Volumkontroll
- Äänenvoimakkuuden säätö
- Ρύθμιση έντασης
- Controlo de volume
- Controlul volumului
- Регулиране на звука
- Hangerőszabályzás
- Garso reguliavimas
- Kontrola glasnoće
- Helitugevuse reguleerimine
- Skajuma regulēšana
- Kontroll tal-volum
- Nastavitev glasnosti
- Ses seviyesi kontrolü

10. Alarm selection
- Alarmkeuze
- Alarmauswahl
- Sélection de l'alarme
- Selección de alarma
- Selezione dell'allarme
- Výběr alarmu
- Wybór alarmu
- Alarmvalg
- Alarmval
- Alarmvalg
- Hälytyksen valinta
- Επιλογή συναγερμού
- Seleção de alarme
- Selectarea alarmei
- Избор на аларма
- Riasztás kiválasztása
- Signalo pasirinkimas
- Odabir alarma
- Häire valik
- Trauksmes izvēle
- Għażla tal-allarm
- Izbira alarma
- Alarm seçimi

Wireless bedwetting alarm system

Quick Guide

Thanks for selecting the Rodger bedwetting alarm

- The Rodger bedwetting alarm is a valuable tool for parents, caregivers, and healthcare professionals to help children overcome bedwetting. Training with the Rodger bedwetting alarm provides an effective and non-invasive method for achieving dry nights.
- The Rodger bedwetting alarm is suitable for children over the age of 5 years.
- The device is not intended for use in the treatment of other types of incontinence or for daytime urinary training.
- The device is not intended to diagnose underlying conditions that may cause enuresis and should be used only after exclusion of other medical causes by a qualified healthcare provider.
- Use the Rodger bedwetting system every night until there have been no "wet beds" for 14 consecutive nights.
- Successful treatment with the Rodger bedwetting system takes from a few weeks to a few months. Persistence is the key to success.
- If your child's bedwetting condition does not improve within 3 months, consult your doctor to rule out any medical conditions.
- Always remove the transmitter immediately from wet sensor pants. This extends the life of the battery and prevents damage to the pants.
- Make sure to remove the transmitter from the pants before washing. The warranty is void if the transmitter has been washed in the washing machine.
- Other transmitters nearby (e.g. wireless baby monitors, garage door openers, weather stations, headsets, etc.) may interfere with the bedwetting alarm system. Change the frequencies of the other devices or remove them.
- Rodger sensor pants are sold separately.

WARNINGS!

- If you are using the AC power supply, please remove the battery from the device.
- This product contains a button cell battery. It is imperative that the battery not be swallowed. If the button cell battery is ingested, it can cause severe internal burns within two hours and may result in death. If there is any suspicion that a battery has been swallowed or has become lodged in a part of the body, immediate medical attention should be sought.
- Any serious incidents should be reported to Rodger BV and to the competent authority of the EU Member State in which user/patient is established.
- It is important to keep new and used batteries out of the reach of children.
- If the battery cover does not close properly, please discontinue use of the product and ensure that it is kept out of the reach of children.
- Please be advised that used batteries must be discarded in accordance with the manufacturer's instructions.
- Please keep transmitter and receiver dry.
- For cleaning the device only use mild detergents and water.

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